

# Liberty<sup>®</sup> Select

united with stay-safe<sup>®</sup>



## Individualized treatment options designed to improve the patient experience.

The Liberty<sup>®</sup> Select peritoneal dialysis (PD) cyclor offers a wide selection of personalized treatment and drain options to meet patients' individual needs.



### Removal of the Scale

This feature promotes a quieter therapy experience for patients and less alarm monitoring for nurses. Note, the Liberty Select cyclor will continue to measure solution volume in the heater bag throughout the treatment.



### Fewer Alarms and Alerts

Flow Alert, which detects if a patient is not draining or filling as expected, can be customized in system settings. Patients can work with their clinicians to adjust audible and/or on-screen Flow Alert notifications on or off based on their preferences. These options are designed to help minimize sleep interruptions and help improve patient experience. Flow Alert occurrences are visible in the IQsystem<sup>®</sup> and Kinexus<sup>™</sup> portal, providing clinicians with timely data to help enable data-driven therapy adjustments. Note, the Flow Alert feature should only be used on software versions 2.9.2 and above.



### More Visibility to Total Ultrafiltration

Setting the "Add Diurnal UF" feature to YES allows staff to easily see patient ultrafiltration from all the drains performed on the cyclor.



### Easier Data Entry and Monitoring

The Patient Data display is blank, making it easier for patients to enter their data daily. This provides better visibility of patient health data to facilitate timely and effective patient management.



### Training Tools

Robust training resources and online learning modules include troubleshooting guides to support clinical evaluations and cyclor operation.

For more information, contact your Fresenius Medical Care Home Therapy Clinical Team.

**INDICATIONS FOR USE:** The Liberty Select cyclor is indicated for acute and chronic peritoneal dialysis. The stay-safe PIN connectors are intended for use with a peritoneal dialysis (PD) cyclor for drainage and infusion of PD solution during PD exchanges. The stay-safe PIN connectors are indicated for use in acute and chronic peritoneal dialysis.

**CAUTION:** Federal (U.S.) law restricts these devices to sale by or on order of a physician.

**INTENDED USE:** The Kinexus Gateway is intended for transmission of patient data collected during peritoneal dialysis therapy with the Liberty Select cyclor.

**NOTE:** Read the Instructions for Use for safe and proper use of these devices. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [fmcna.com](http://fmcna.com).

© 2021 Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Liberty, Liberty Select, IQsystem, Kinexus, and stay-safe are trademarks of Fresenius Medical Care Holdings, Inc., or its affiliated companies. All other trademarks are the property of their respective owners. P/N 103958-01 Rev B 05/2021



**FRESENIUS  
MEDICAL CARE**