PRODUCT NOTIFICATION

DIASAFE[®]*plus*_{US}

Date: September, 2025

Dear Valued Customer:

I am pleased to notify you about the introduction of the DIASAFE*plus*_{US} dialysis fluid filter (P/N F00007039) for use on Fresenius Medical Care 2008[®] Series Hemodialysis Machines equipped for use with the DIASAFE*plus* filter.

The DIASAFE*plus*_{US} filter will replace the current DIASAFE*plus* dialysis fluid filter (P/N 5008203) which will no longer be available in the U.S. This filter is an identical product to the current DIASAFE*plus* dialysis fluid filter (P/N 5008203) but the Instructions for Use is specific to the U.S. The product is still packaged with 10 filters per case and there is no change in pricing.

Orders for the DIASAFE*plus* dialysis fluid filter (P/N 5008203) will automatically be changed to the DIASAFE*plus*_{US} dialysis fluid filter (P/N F00007039) as product stock transitions.

Please note the following important information when using the new DIASAFE*plus*_{US} dialysis fluid filter:

1. Exchange intervals

- **Note:** Performing chemical disinfection more frequently than recommended below increases wear and membrane degradation.
- When weekly chemical disinfection is performed, the DIASAFE*plus*_{US} filter exchange interval is a maximum of 90 days.
- When chemical disinfection is performed more than weekly, the DIASAFE*plus*_{US} filter is required to be exchanged after a maximum of 30 days usage and not exceeding 30 chemical disinfection cycles.
- The DIASAFE*plus*_{US} filter must be exchanged immediately when a leak is detected or if the Diasafe/Pressure Test fails.
- Either a **Heat or Chemical Disinfection** must be performed after a new DIASAFE*plus*_{US} filter is installed. **Note:** The machine must pass the Diasafe/Pressure test after installing a new DIASAFE*plus*_{US} filter before performing Heat or Chemical Disinfection.
- A chemical disinfection should be performed using a bleach (Sodium Hypochlorite NaOCl) solution with a concentration not exceeding **6.15%**.

-
- Refer to the **“Replacing the DIASAFE Filter”** section in Chapter 6, **“Alarms and Troubleshooting”**, of the Fresenius Medical Care 2008 Series Hemodialysis Machine Operator’s Manual for instructions on exchanging and installing a DIASAFE*plus*_{US} filter.

2. Cleaning and disinfection

- Cleaning and disinfection of the DIASAFE*plus*_{US} filter is part of the cleaning and disinfection process for the Fresenius Medical Care 2008 Series Hemodialysis Machine hydraulics.
- Refer to Chapter 5 “Disinfection and Maintenance” of the Fresenius Medical Care 2008 Series Hemodialysis Machine Operator’s Manual for cleaning and disinfection instructions.
- The latest Fresenius Medical Care 2008 Series Hemodialysis Machine Operator’s Manual is available for download at www.fmcna.com.
- The acceptable solutions for performing “Acid Clean” for the Fresenius Medical Care 2008 Series Hemodialysis Machine are listed below:
 - 2-5% Citric Acid
 - 5% Acetic Acid (white distilled vinegar)
- Bleach (Sodium Hypochlorite NaOCl) is the acceptable and recommended solution for chemical disinfection. The maximum strength of bleach used should not exceed **6.15%**.

3. Warnings

A section with warnings has been introduced in the DIASAFE*plus*_{US} filter Instructions for Use.

Please note, that any failure to follow the warnings provided may increase the risk of patient exposure to bacteria and/or endotoxin. This applies especially but not limited to the cases listed below:

- If the initial and/or regular disinfection of the DIASAFE*plus*_{US} filter is not performed according to the instructions in Chapter 5 **“Disinfection and Maintenance”** of the Fresenius Medical Care 2008 Series Hemodialysis Machine Operator’s Manual.
- If the DIASAFE*plus*_{US} filter is used beyond the recommended exchange intervals.
- If the quality/purity of the feed water does not meet the requirements e.g. due to insufficient maintenance of the water system.

4. Other changes:

- Addition of the “US” subscript after the DIASAFE*plus* name.
- Creation of a new part number, P/N F00007039, which replaces the current P/N 5008203.
- Addition of instructions on how to perform a filter exchange

Please contact Technical Support at 1-800-227-2572 for further information.

As the industry's leading provider of hemodialysis equipment, we are committed to providing the highest levels of equipment performance, support, and service. Thank you for your continued support of Fresenius Medical Care hemodialysis products.

Sincerely,

Robert Gregory
Product Director - Hemodialysis Machines and Spare Parts

**Please forward this letter to the Medical Director,
Biomedical Managers and Staff, Nurses and Technicians**

Attachment: P/N 7531851 IFU DIASAFE*plus*_{US}

Indications for Use: The DIASAFE*plus*_{US} is intended for the preparation of ultrapure dialysate and sterile, non-pyrogenic substitution fluid from pretreated water and is not intended to provide primary purification. Attention must still be given to the chemical and microbiological quality of water and concentrates and the maintenance of supply systems (e.g. RO system, central delivery system). The microbiological quality (microbial count [CFU/ml] and endotoxin measurement [EU/ml]) of the incoming water should be < 200 CFU/ml and < 2 EU/ml, respectively.

The DIASAFE*plus*_{US} can only be used with Fresenius Medical Care dialysis machines fitted for use with DIASAFE*plus*_{US}.

DIASAFE[®] *plus*_{US}

Dialysis Fluid Filter

Technical Data

Filtration rate	≥ 300 ml/h·mmHg ≥ 3.75 l/min·bar
Maximum filtration pressure	2 bar
Membrane material Housing material Potting material Sealings Connection	Fresenius Polysulfone [®] Polypropylene Polyurethane Silicone DIAFIX [™] Lock System
Surface area	2.1 m ²
Dialysis fluid purity	< 0.03 EU/ml, < 0.1 CFU/ml (ultrapure dialysate)
Filter Exchange	
5008X machines	2008-series machines
▪ The maximum life of the filter (90 days) has been reached ▪ The maximum number of chemical heat disinfection (100) has been reached. ▪ The maximum number of chemical/rinse (cold disinfection) (13) has been reached ▪ If the filter is defective (T1 test failed) ▪ If an external leak is present ▪ If the orange installation date label is missing or not readable	▪ Every 90 days when a weekly chemical disinfection schedule is followed ▪ After 30 days, if more frequent chemical disinfections than above are performed. Do not exceed 30 chemical disinfections without exchanging the DIASAFE[®] <i>plus</i>_{US} ▪ If DIASAFE test fails ▪ If an external leak is present ▪ If the orange installation date label is missing or not readable
Art. No.	F00007039

Caution: Federal law restricts this device to sale by or on the order of a physician

**FRESENIUS
MEDICAL CARE**

Manufactured for Fresenius Medical Care RTG LLC
920 Winter Street
Waltham, MA 02451
+1-800-323-5188

DIASAFE[®]*plus*_{US}

Dialysis Fluid Filter

GENERAL INFORMATION

Refer to product or carton label for:

 Expiry date	 Refer to instructions for use
 Batch	 Order number
 Date of manufacture	 ^{+30°C} _{+5°C} Storage temperature range

Indications for Use: The DIASAFE[®]*plus*_{US} is intended for the preparation of ultrapure dialysate and sterile, non-pyrogenic substitution fluid from pre-treated water and is not intended to provide primary purification. Attention must still be given to the chemical and microbiological quality of water and concentrates and the maintenance of supply systems (e.g. RO system, central delivery system).

The microbiological quality (microbial count [CFU/ml] and endotoxin measurement [EU/ml]) of the incoming water should be < 200 CFU/ml and < 2 EU/ml, respectively.

The DIASAFE[®]*plus*_{US} can only be used with Fresenius Medical Care dialysis machines fitted for use with DIASAFE[®]*plus*_{US}.

Failure to follow the safety relevant warnings below may increase the risk of patient exposure to bacteria / endotoxin.

For 5008X machines:

WARNING: After installation of a new DIASAFE®*plus*_{US}, disinfection must be performed

WARNING: Adequate filter disinfection is the responsibility of the user. Filter disinfection beyond what is recommended in the Operating Instructions of the dialysis machine could result in premature membrane degradation.

WARNING: DIASAFE®*plus*_{US} must be exchanged:

- The maximum life of the filter (90 days) has been reached
- The maximum number of chemical heat disinfection (100) has been reached
- The maximum number of chemical/rinse (cold disinfection) (13) has been reached
- If the filter is defective (T1 test failed)
- If an external leak is present
- If the orange installation date label is missing or not readable

WARNING: The selection and maintenance of water pre-treatment equipment for dialysis is the responsibility of the clinic – the water quality should be tested according to clinic policy. Use water needs to be < 200 CFU/ml for microbial count and < 2 EU/ml for endotoxin measurement.

For 2008-series machines:

WARNING: After installation of a new DIASAFE®*plus*_{US}, heat or chemical disinfection must be performed.

WARNING: The mandatory DIASAFE and Pressure Test must be performed in accordance with part E in section “Exchange / Installation”.

WARNING: Adequate filter disinfection and acid cleaning is the responsibility of the user. Chemical filter disinfection or acid cleaning beyond what is recommended in the Operating Instructions of the dialysis machine could result in premature membrane degradation.

WARNING: DIASAFE®*plus*_{US} must be exchanged

- Every 90 days when a weekly chemical disinfection schedule is followed
- After 30 days, if more frequent chemical disinfections than above are performed. Do not exceed 30 chemical disinfections without exchanging the DIASAFE®*plus*_{US}.
- If DIASAFE test fails
- If the filter shows an external leak
- If the orange installation date label is missing or not readable

WARNING: The selection and maintenance of water pre-treatment equipment for dialysis is the responsibility of the clinic – the water quality should be tested according to clinic policy. Use water needs to be < 200 CFU/ml for microbial count and < 2 EU/ml for endotoxin measurement.

Disinfection / Cleaning

DIASAFE®*plus*_{US} is included in all machine cleaning and disinfection cycles. Disinfection according to the operating instructions is essential. Please refer to the Operating Instructions of the dialysis machine for further details.

Exchange / Installation

Note: Check expiration date prior to use

For 5008X machines:


Refer to the Operating Instructions of the dialysis machine for further details on exchange/installation instructions on the 5008X machine.

For 2008-series machines:

Refer to the instructions below for exchange/installation on the 2008-series machines:

DIASAFE[®]plus_{US} must only be installed in a specially designed filter holder called the DIAFIX[™]Lock System. This lock system ensures the safe connection between the filter and the hydraulic components of the machine. Following new filter installation, heat or chemical disinfection is mandatory.

- A. Turn off the hemodialysis machine prior to conducting a DIASAFE[®]plus_{US} filter exchange / installation.
- B. Remove the clear protective cover from the DIAFIX[™]Lock System, exposing the DIASAFE[®]plus_{US} filter.
- C. Remove old DIASAFE[®]plus_{US} filter
 1. Lift up the lock levers on the left side of the filter mount and slide the used DIASAFE[®]plus_{US} up and out.
 2. Follow your clinic procedures for filter disposal.
- D. Install new DIASAFE[®]plus_{US} filter (also refer to the illustration in figure 1 below)
 1. Remove the plastic tabs from a new DIASAFE[®]plus_{US} filter.
 2. Place the filter in the two grooves at the top and the bottom of the mount.
 3. Slide the filter down until it clicks into place.
 4. Push the lock levers down to secure the filter into the mount.
 5. Place the orange product installation date label on the filter.
 6. Install the clear protective cover.
- E. Test and disinfect new DIASAFE[®]plus_{US} filter
 1. Turn on the hemodialysis machine and select dialysis mode.
 2. Perform a Pressure holding test and a DIASAFE test after filter change.
 3. After passing the Pressure holding and DIASAFE Tests, perform a heat or chemical disinfection according to clinic procedure.

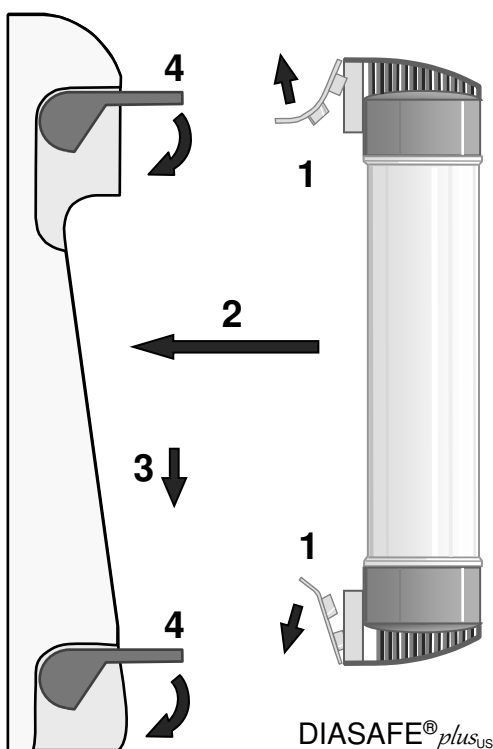DIAFIX[™]Lock System

Figure 1: Installation of the DIASAFE[®]plus_{US}

WARRANTY

Products with manufacturing defects will be replaced if the defect is reported with details of the lot number.

The manufacturer shall not be liable for any misuse, improper handling, non-compliance with instructions for use and cautionary notes and for any damage incurred subsequent to the manufacturer's delivery of the DIASAFE® *plus*_{US}.

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