



## Fresenius Medical Care North America Service Bulletin

Equipment: 2008K@Home Hemodialysis Machines  
Bulletin: 20-HOM-001 Rev A  
Subject: bibag® Disposable Installation Update

### 1.0 PURPOSE

To provide an update to the installation of a bibag® disposable onto the bibag® connector on bibag® equipped 2008K@Home™ Hemodialysis Machines.

### 2.0 SCOPE

As part of continuous improvement, the 2008K@Home™ User Guide (P/N 490180) is being updated to provide clarity and capture accurate intent of how to connect a bibag® disposable to the bibag® connector nozzles. The change applies to step (c) of the note listed below:



**Note:** If the bibag disposable is the bicarbonate source, do not pull the blue bicarbonate connector from the machine's rinse port. Do the following to prepare the bibag disposable:

- Remove the plastic seal from underneath the water and bicarbonate nozzles of the bibag disposable.
- Open the bibag door on the machine by lifting up on the dark-gray handle.
- With the white bibag handle facing outward, hang the bag on the bibag connector nozzles.
- Close the door, making sure it latches firmly in place. An audible click means the door is closed.

With the white bibag® handle facing outward, hang bibag® disposable on the bibag® connector nozzles. Push down where indicated in the figure below (red arrows) until it is fully seated onto the bibag® connector nozzles.

Once the bibag® disposable is fully seated, close bibag® door until it latches into place (*an audible click is heard when door is closed*)



If there are any questions regarding this bulletin, contact Fresenius Medical Care Technical Support at 800-227-2572.

© 2020 Fresenius Medical Care North America. All Rights Reserved.