



Fresenius Medical Care North America Service Bulletin

Equipment: All 2008[®] Series Hemodialysis Machines
 Bulletin: 21-FRS-002 Rev B
 Subject: Sensor Board Compatibility

1.0 PURPOSE

To inform all Field Service and Technical Support personnel as well as Fresenius Medical Care-trained customers of Sensor board compatibility in the 2008[®] Series Hemodialysis machines.

2.0 SCOPE

Currently, there are three types of Sensor boards (part numbers 190332, 191142 & 191191) for the 2008[®] Series Hemodialysis machines.

Below are the required Actuator-Test board software versions for each type of machine and the appropriate Sensor board part number:

Machine Type	Actuator-Test Board Part Number	Actuator-Test Software Version	Sensor Board Part Number
2008 [®] K	190234 only	Any version	190332 only
2008 [®] K ²	190234 only	Any version	190332 only
2008K@Home™	190234 only	Any version	190332 only
2008 [®] T	190234 or 191203	Versions less than 2.32	190332 only
2008 [®] T	190234 or 191203	Versions 2.32 or greater	190332, 191142 or 191191
2008T BlueStar™ Currently equipped with a 190332 Sensor board	190234 or 191203	Versions 2.32 or greater	190332, 191142 or 191191
2008T BlueStar™ Currently equipped with a 191142 or 191191 Sensor board	190234 or 191203	Versions 2.32 or greater	191142 or 191191

An option is available so a 191142 or 191191 Sensor board can be used in a non-2008T BlueStar™ machine. The 2008T BlueStar™ Standard Kit (part number 191137) will install the appropriate software versions onto the Functional and Actuator-Test boards.

If there are any questions regarding this bulletin, contact Fresenius Medical Care Technical Support at 800-227-2572.

© 2021 - 2022 Fresenius Medical Care. All Rights Reserved.