

# Quick Start: stay•safe® CAPD System

## Instructions

Use the following checklist to ensure the proper steps are taken. Use aseptic technique throughout this procedure. Check the box to mark each step performed.

✓	STEPS
	Prepare work area
	Gather supplies
	Warm solution bag
	Position organizer on clean work surface or on IV pole holder
	Expose your catheter extension set and ensure white clamp is closed
	Mask, then wash hands
	Open solution bag outer wrap
	Check solution bag for S.C.A.L.E.
	Locate stay•safe DISC
	Turn blue dial counterclockwise until it fits into the cutout
	Remove plastic cover
	Place DISC into the organizer
	Press tubing into the channels
	Hang solution bag
	Place drain bag on floor
	Break the cone
	Remove stay•safe cap from package and insert into “New Cap” notch
	Place end of catheter extension set into “Extension” notch
	Clean hands
	Unscrew connector cap from stay•safe DISC
	Unscrew catheter extension set from stay•safe cap and immediately connect it to the DISC



✓	STEPS
	Place removed connector cap from stay•safe DISC onto used cap and discard
	Open catheter extension set clamp
	Drain will take about 20–30 minutes
	Turn blue dial clockwise to position 2: Flush
	After a few seconds, turn dial to position 3: Fill
	Fill will take about 10–15 minutes
	Turn blue dial clockwise to very end of position 4: PIN/Close
	Close clamp on catheter extension set
	Clean hands if instructed to do so
	Unscrew protective cover from new stay•safe cap
	Unscrew extension set from the DISC
	Immediately connect catheter extension set with PIN to new stay•safe cap
	Remove capped extension set from the organizer
	Secure catheter extension set to abdomen
	Discarding of CAPD double bag set:
	Connect removed protective cover from the used stay•safe cap to open end of DISC
	Check and weigh drained dialysate
	Remove the whole stay•safe system from IV pole and organizer
	Dispose of solution

Read the Instructions for Use for safe and proper use of this device. For a complete description of hazards, contraindications, side effects, and precautions, see full package labeling at [fmcna.com](http://fmcna.com).

## Indications for Use

The CAPD stay•safe Disposable Administrative Sets with stay•safe Connector is indicated for use in end stage acute and chronic renal disease. DELFLEX® peritoneal dialysis solutions are indicated in the treatment of chronic renal failure patients being maintained on peritoneal dialysis when nondialytic medical therapy is judged to be inadequate.

## Caution

Federal (US) law restricts this device to sale by or on the order of a physician.

© 2020 Fresenius Medical Care. All Rights Reserved. Fresenius Medical Care, the triangle logo, Fresenius Renal Technologies, DELFLEX, and stay•safe are trademarks of Fresenius Medical Care Holdings, Inc. or its affiliated companies. All other trademarks are the property of their respective owners. P/N 104448-01 Rev A 12/2020



**FRESENIUS  
MEDICAL CARE**

RENAL THERAPIES GROUP